

安徽宏宇五洲医疗器械股份有限公司

ANHUI HONGYU WUZHOU MEDICAL MANUFACTURER CO.,LTD

Add: No.2, Guanyin Road, Economy Development Zone, Taihu, Anhui Province, China

Инструкция по применению медицинских изделий

Одноразовый шприц { 1 мл, 2 мл, 2,5 мл, 3 мл, 5 мл, 10 мл, 20 мл, 30 мл, 50 мл, 60 мл, 100 мл, 150 мл } с иглой { 0,3 (30G), 0,33 (29G), 0,36 (28G), 0,4 (27G), 0,45 (26G), 0,5 (25G), 0,55 (24G), 0,6 (23G), 0,7 (22G), 0,78 (21G), 0,9 (20G), 1,1 (19G), 1,2 (18G), 1,5 (17G), 1,6 (16G), 1,8 (15G) } предназначен для одноразового использования, стерильный, нетоксичный, апиrogenный.

Состоит из цилиндра, плунжера, резинового поршня, ступицы, канюли, колпачка.

Способ применения:

1. Вскройте индивидуальную упаковку и снимите колпачок
2. Наберите лекарственный препарат иглой вверх, медленно надавите на поршень, чтобы убрать воздух
3. Произведите инъекцию

Внимание:

1. Стерильно пока упаковка не вскрыта или повреждена
2. Не используйте если колпачок не на своем месте
3. Избегайте замерзания или чрезмерного нагревания
4. Изделие необходимо использовать сразу после вскрытия, после использования сразу выбросить
5. Только для однократного применения. Не стерелизовать и не использовать повторно

Срок годности и температура хранения:

Срок годности изделий со дня изготовления - три года, изделия должны реализоваться до исхода срока годности. Номер партии и срок годности нанесены на маркировке индивидуальной упаковки.

Данное изделие необходимо хранить при влажности не более 80%, рекомендуемая температура хранения должна быть от -5 °C до + 30 °C, без агрессивных газов, в прохладном, сухом, хорошо вентилируемом и чистом помещении. На складе должны предприниматься меры профилактики и защиты от вредителей

安徽宏宇五洲医疗器械股份有限公司

ANHUI HONGYU WUZHOU MEDICAL MANUFACTURER CO.,LTD

Add:No.2.Guanyin Road,Economy Development Zone, Taihu, Anhui Province,China

INSTRUCTION FOR USE OF MEDICAL DEVICE

The disposable syringe { 1ml , 2ml , 2.5ml , 3ml , 5ml , 10ml , 20ml , 30ml , 50ml , 60ml , 100ml , 150ml } with needle { 0.3(30G) , 0.33(29G) , 0.36(28G) , 0.4(27G) , 0.45(26G) , 0.5(25G) , 0.55(24G) , 0.6(23G) , 0.7(22G) , 0.8(21G) , 0.9(20G) , 1.1(19G) , 1.2(18G) , 1.5(17G) , 1.6(16G) , 1.8(15G) } is for single use, Sterile, Non Toxic Non pyrogenic.

And is constructed by barrel , plunger , rubber piston , hub , cannula ,cap.

Direction of use:

- 1.Open individual packing and remove the cap.
- 2.Pumped drug liquid, the needle upwards, slowly push the plunger to exclude air
- 3.Hypodermic injection

Caution:

- 1.Sterile unless package has been opened or damaged
- 2.Do not use if tip protectors are not in place
- 3.Avoid freezing and excessive heat.
- 4.The product should be used immediately after envelope opened, discard after use.
- 6 For single use only. Do not resterilize or re-use.

Shelf life and storage temperature:

Production from the date of production is valid for three years , must be used within the period of validity. The batch number and expiry date was printing on the seal of individual packing.

This product should be stored in the relative humidity of not more than 80%, suggested storage temperature should be +5°C to +30°C, no corrosive gas , a cool , dry ,well ventilated and clean room. Storage warehouse should be prevention and pest prevention measures.

ANHUI HONGYU WUZHOU MEDICAL MANUFACTURER CO., LTD.
安徽宏宇五洲医疗器械股份有限公司

Anhui Hongyu Wuzhou Medical Manufacturer Co., Ltd.

2015.07.08

张洪瑜

